

Opening The Mini Original For Cleaning

A 5/32" Alan key (hex key) is required. A pair of gloves may be helpful to grip the emptier. Remove the end screw and remove the valve body and stem from the coil and support. Insert the Alan key in the end of the stem as shown in the picture.



Grasp the valve body with one hand and the Alan key with the other.



Twist rapidly in opposite directions to unlock the seal



Unscrew the stem from the valve body



Take care to not drop the plunger and spring on the floor.



Separate the parts and remove any debris present.



Reassemble by reversing the steps.